Almohadilla Eléctrica Súper

#13022
30,5 x 38 cm

Temperaturas

Doble Termostato

Funda Desmontable

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MANUAL DE INSTRUCCIONES

Sistema de Calidad ISO 9001

Instrucciones Importantes de Uso:

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VENTA LIBRE

Important Instructions for Use:

1- Read the following instructions carefully before using the heating pad and keep them to ensure a proper future use.

2- This pad has a switch with a sliding key that admits three well-defined positions. These positions allow a temperature selection adaptable to your needs. .

In the **second position** of the key, the temperature is **maximum**. It is advised to start using the pad in the second position, and once the pad starts to heat, select the first position (depending on your need).

3- Before each use carefully check the inner cover and the wire to make sure there are no damages nor worn out pieces. If they are detected, do not use the heating pad and contact the after sales service center.

4- In order to get an excellent long-term performance and to avoid risks arising from the product misuse, we highly recommend to follow these instructions for use:

- This pad is water proof, **do not use the heating pad if it is wet**. Bear in mind it operates on household current.
- **Do not sit on the pad, do not crush nor fold it**, (place the pad where heat is applied on the body and not under it).
- **Do not use it without the cloth cover**, do not pull the pad by the cord and do not use the cord as a handle.
- **Never use pins, needles or any other sharp metal object** to hold the pad into the desired position.

- Do not use it on children, handicapped or sleeping people or those with poor blood flow.

Do not perform the following actions due to possible burnings:

- Do not use the pad while sleeping nor cover it with blankets. (Burns may be caused regardless of control setting)
- Do not apply cream or liniment, they may cause more heat than the recommended.

Do not use the pad at the maximum temperature settings

- Check the skin under the pad frequently to avoid burns and blisters.

- Unplug the pad after being used - without exception-

The device must not be used by people who are mentally/ physically unable, with reduced sensory capacity, or does not have the experience or knowledge, unless they are supervised or instructed by a person in charge of their safety.

Children must be supervised while using the device in order to prevent them from playing with it.

Cleaning and Maintenance:

1- The cloth cover may be dry-cleaned or hand-washed using soap and cold water (do not squeeze it). Remember to **remove the cover from the pad** before this operation.

2- Put the heating pad away without pressing on it and avoiding crumples.

Technical specifications:

- ✓ 220 V - 50 Hz - 55W
- 3 Temperatures

Composition: inner cover 100% PVC, cloth cover 100% polyester.

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-Las reparaciones se realizarán únicamente en los domicilios de los Servicios Autorizados por San Up S.A.