SAN-UP®

Almohadilla Eléctrica Súper Plus

#13133
30,5 x 38 cm

Temperaturas

Doble Termostato

Funda Desmontable

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MANUAL DE INSTRUCCIONES

Sistema de Calidad ISO 9001

Instrucciones Importantes de Uso:

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Garantía: 1 año

Fabricado por: EAGLE ELECTRIC (SHEN ZHEN) CO., LTD

Huangfengling N° 4 Industrial Zone, Shiyan Town, Ban'an Distric,

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VENTA LIBRE

Important Instructions for Use:

1- Read instructions carefully before using the heating pad. Keep them to ensure proper use of the product.

2- This heating pad has a switch with 4 preset temperatures:

- 1- minimum
- 2- low
- 3- medium
- 4- maximum

It is also provided with a 1-hour automatic cut-off.

It is recommended to start using the heating pad at **position 2**, and once it is heated select the temperature desired. To start using the device press the settings switch button, a red light will come on at the **minimum setting**. If you wish to select a different setting, keep pressing the button until the light of the position desired comes on. To turn the device off, press until no light is on.

3- Before each use, carefully check the PVC internal body and cord for signs of wear or tear. In case of wear and tear, do not use the heating pad and contact the **technical service authorized by SAN UP**.

4- In order to ensure optimum operation for many years and to reduce the risks arising from misuse, we recommend using the **heating pad as follows**:

The heating pad is not moisture-proof, **do not use it if wet** (please note that for the device to operate it must be connected to the mains).

- **Do not sit, crush or bend the heating pad**, just place it on the part of the body to be heated (never underneath).

- **Do not use it without the cover**, or pull the cord or use it as a handle,

- **Never use pins or sharp objects** to fasten it in the position of use.

- The device must not be used by people who are mentally/ physically unable, with reduced sensory capacity, or does not have the experience or knowledge, unless they are supervised or instructed by a person in charge of their safety.

Children must be supervised while using the device in order to prevent them from playing with it.

- **Avoid doing the following since it could result in burns:**

Do not use the device while asleep, or cover it with a blanket (regardless of the selected temperature setting)

Do not use the device with liniments or creams, since they produce temperatures higher than the recommended ones. Do not use it at maximum temperature for long periods.

- **Check the skin condition frequently to avoid burns or blisters.**

- **Without exceptions, disconnect the heating pad after each use.**

Cleaning and Maintenance:

1- **The outer cover can be dry-cleaned or washed** with cold water and mild soap (dry it without crumpling). Before clearing, separate the outer cover from the PVC internal body.

2- **Store the heating pad without exerting too much pressure, and avoiding bends and creases.**

Technical specifications:

✓ 220V-50Hz - 55W

✓ 4 Temperatures

Components: Inner cover: 100% PVC, outer cover: 100% polyester.

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-Las reparaciones se realizarán únicamente en los domicilios de los Servicios Autorizados por San Up S.A.