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
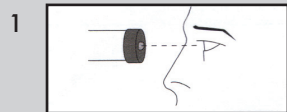
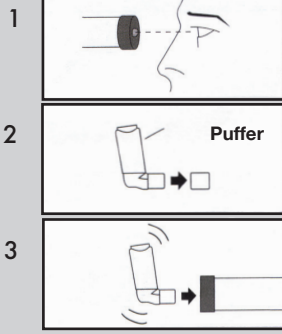
## Instructions for use

Before you use the inhalation chamber for the first time, wash according to the cleaning instructions described below:

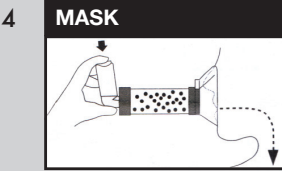1 Read instructions supplied with your inhaler (Puffer).  
Visually check the inhalation chamber for foreign objects.


2 Remove protective cap from your Puffer

3 Shake your Puffer before each use, and insert into the backpiece of the inhalation chamber.

**4 MASK**

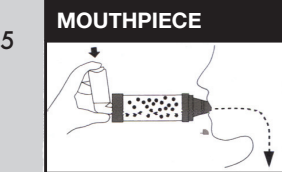**For mask users:**  
Place the mask gently to the face so that the mouth and nose are covered. Be certain to create a good seal between mask and face. Depress Puffer at beginning of slow inhalation. Always administer only ONE puff at a time.  
**For babies and children:** Maintain seal for 5 - 6 breaths after depressing the Puffer.  
**For adults:** Maintain seal for 2 - 3 breaths after depressing the Puffer.


**5 MOUTHPIECE**

**For mouthpiece users:**  
Place mouthpiece in the mouth ensuring a tight seal with the lips. Depress your Puffer at beginning of slow inhalation. Always administer only ONE puff at a time.  
Inhale deeply and hold as long as possible, up to 10 seconds before breathing out. An alternative is to keep mouth tight on the mouthpiece and breathe slowly 2-3 times after Puffer is depressed.


Follow the instructions supplied with your Puffer on the time period to wait between PUFFS before repeating step 4 or 5.
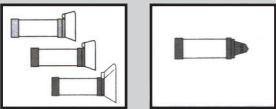
## Cleaning instructions:

The inhalation chamber should be cleaned before first use, and then weekly.

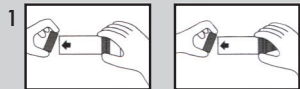**Caution:**  
Running water through the inhalation chamber may harm the one-way valve. NEVER DISASSEMBLE THE MASK OR THE ONE-WAY VALVE as these parts may become loose if handled and may affect the overall reliability of the product.

1 Remove only the backpiece of the inhalation chamber..

2 Soak the parts in lukewarm water for 20 minutes using a mild detergent, and gently agitate.

3 Do not rinse. Shake out excess water droplets. Do not dry or rub with a cloth.

4 Let air dry in a vertical position.  
Ensure all parts are completely dry before reassembly.

5 Re-attach backpiece to the chamber.

## Cautions

- This device should only be used for the purpose for which it has been prescribed. The manufacturer is not responsible for damages caused by inappropriate or incorrect use.
- This product should only be used as instructed with medications which have been prescribed by your healthcare professional.
- Before using the inhalation chamber it is important to read the instructions.
- This product should not be sterilized or washed in the dishwasher.
- Do not leave the inhalation chamber unattended with children.
- Single patient use only.

## Notes

- Read and follow the patient instructions provided with your Puffer.
- Do not spray MORE THAN ONE PUFF AT A TIME into the inhalation chamber before or during an inhalation.
- For best results the patient should breath through their mouth.
- This product is latex and ftalatos free.
- The inhalation chamber may need to be replaced afer 6-12 months use.

## Sizing Guide

Baby Mask	Child Mask	Adult Mask/Mouthpiece
0-12 months	12 months - 5 yrs.	5 yrs. and older