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MINI # 3262

PRECAUCIONES IMPORTANTES:

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MINI # 3262

IMPORTANT SAFEGUARDS:



When using an electrical appliance, basic precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING

- Close supervision is necessary when this appliance is used by, on or near children, invalids or disabled persons.
- Use this appliance only for its intended use as described in this manual. Do not use any attachments not provided with the unit.
- Do not place or store appliance where it can fall or be dropped into a tub or sink.
- Do not place or drop into water or any other liquid.
- This unit should not be used by children without adult supervision. THIS IS NOT A TOY.
- Turn off appliance when not in use.
- Never operate this appliance if it has been damaged or if it is not working properly.
- Keep away from heated surfaces.
- This product is a non-professional appliance designed to provide soothing massage to worn muscles. Do not use this product as a substitute for medical attention.
- This appliance is designed for household use only.

BATTERY PRECAUTIONS

- Use only the size and type of batteries specified.
- When installing batteries, observe proper +/- polarities. Incorrect installation of battery may cause damage to the unit.
- Do not mix different types of batteries together (e.g. alkaline with carbon-zinc or old batteries with new ones).
- If the unit is not going to be used for a long period of time, remove batteries to prevent damage due to possible battery leakage.
- Do not dispose of batteries in fire. Batteries may explode or leak.

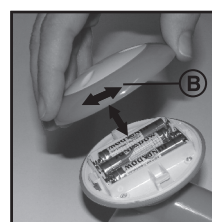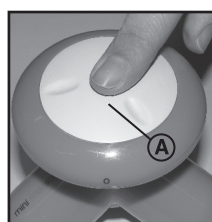OPERATING INSTRUCTIONS

- Press (A) to activate MINI.
- Hold MINI over desired area to be massaged.
- After completing your massage, turn MINI off by depressing.
- When batteries need replacing, open battery compartment (B) by unscrewing top counter clockwise. Insert three "AAA" batteries into the chamber as directed on the bottom of the compartment and replace top.

Imported, distributed
and guaranteed by

Made in China.